



Acce/ Prevail Chemosterilant & High Level Disinfectant

Chemosterilant, for use on all Foot Care related devices compatible with Hydrogen Peroxide such as nail nippers and splitters, scissors, files, probes, nail elevators and burs.

SPORICIDAL: 6hrs

TUBERCULOCIDAL: 10 Min.

FUNGICIDAL: 10 Min.

DIRECTIONS FOR STERILIZATION of CRITICAL ITEMS & HIGH LEVEL DISINFECTION of SEMI-CRITICAL ITEMS.

CRITICAL ITEMS: Chemosterilization for instruments and devices that come in contact with sterile tissue, the blood stream & recirculating body fluids - such as surgical instruments and foot care instruments. E.g.: Cuticle Cutters, Corn & Callous Slicer.

Place pre-cleaned, rinsed and rough dried instruments into a container of undiluted **ACCEL**. Allow the instruments to soak for at least 6 hours. Remove and rinse with sterile water.

SEMI-CRITICAL ITEMS: High-Level Disinfection for items that come in contact with non-intact skin or mucous membranes such as, Nail Cutters, Cuticle pushers, Nail Scissors.

Place pre-cleaned, rinsed and rough dried instruments into a container of undiluted **ACCEL**. Allow the instruments to soak for at least 10 minutes. Remove and rinse with sterile or potable water.

14-DAY RE-USE CLAIM: This solution may be used and re-used for up to 14 days. Solution in its original packaging is good until date of expiry.