



## **ACCEL Wash Instrument Cleaner**

Instrument cleaning solution. May be used for pre soak, manual, ultrasonic cleaning.

### **How To Use**

**MANUAL CLEANING:** Dilute Accel Wash 1:128 with warm water. Place instrument into the solution for 5 minutes using agitation when required. Remove instrument, rinse, inspect for cleanliness and proceed with disinfecting step. Change solution daily or as needed.

**ULTRASONIC** Dilute Accel Wash 1:128 with warm water. Place instrument into the solution for 3-5 minutes. Remove instrument, rinse, inspect for cleanliness and proceed with disinfecting step. Change solution daily or as needed.